


## System requirements for sona:fit

### Check list

System requirements	Minimum	Recommended	Meet requirements yes/no
Processor type & speed:	Pentium III, 1 GHz or similar	Pentium IV, 2GHz or faster	
Manufacturer:			
System RAM:	512 MB	1 GB or more	
Available disk space:	1 GB	1 GB or more	
Operating system:	Windows 7, Windows Vista (Edition: Home, Business, Enterprise, and Ultimate) (SP1), Windows XP (SP2/SP3)	Windows XP Professional (SP2/SP3)	
Screen resolution:	1024x768 panel		
CD-ROM drive:	Required		
Serial com port:	One for HI-PRO (can be replaced by a USB Serial Com port adapter; alternatively HI-PRO USB)		
USB port:	For HI-PRO USB		
HI programming interface:	HI-PRO or HI-PRO USB	NOAHlink	
Noahlink driver	NOAHlink 1.50.04 NOAHlink 1.50.03	NOAHlink 1.54.01 NOAHlink kernel 1.52.00	
Noah	Version 3.6.1	Version 3.6.1 or higher	
Internet Explorer	Version 5.01	Version 6.0 or higher	
Internet at fitting station	Version 5.01	Version 6.0 or higher	


## Quick check for Operating System


- : Click Start
- : Highlight My Computer & right click
- : Select Properties

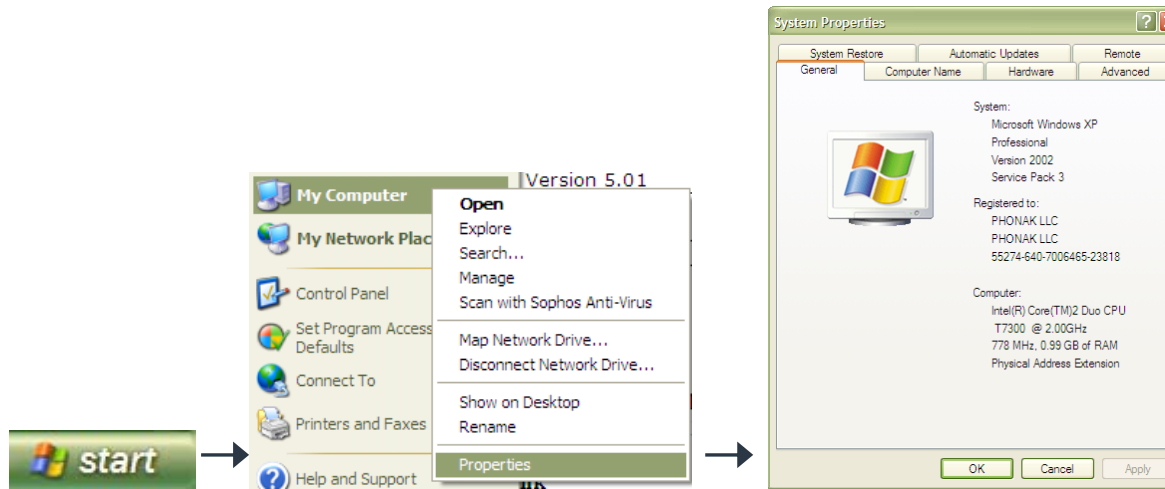

**Quick check for NOAHlink (NOAHlink must be connected):**


- : Click Start
- : Select NOAHlink
- : Select NOAHlink Properties
- : Click Settings tab
- : Click Advanced
- : Click Version tab

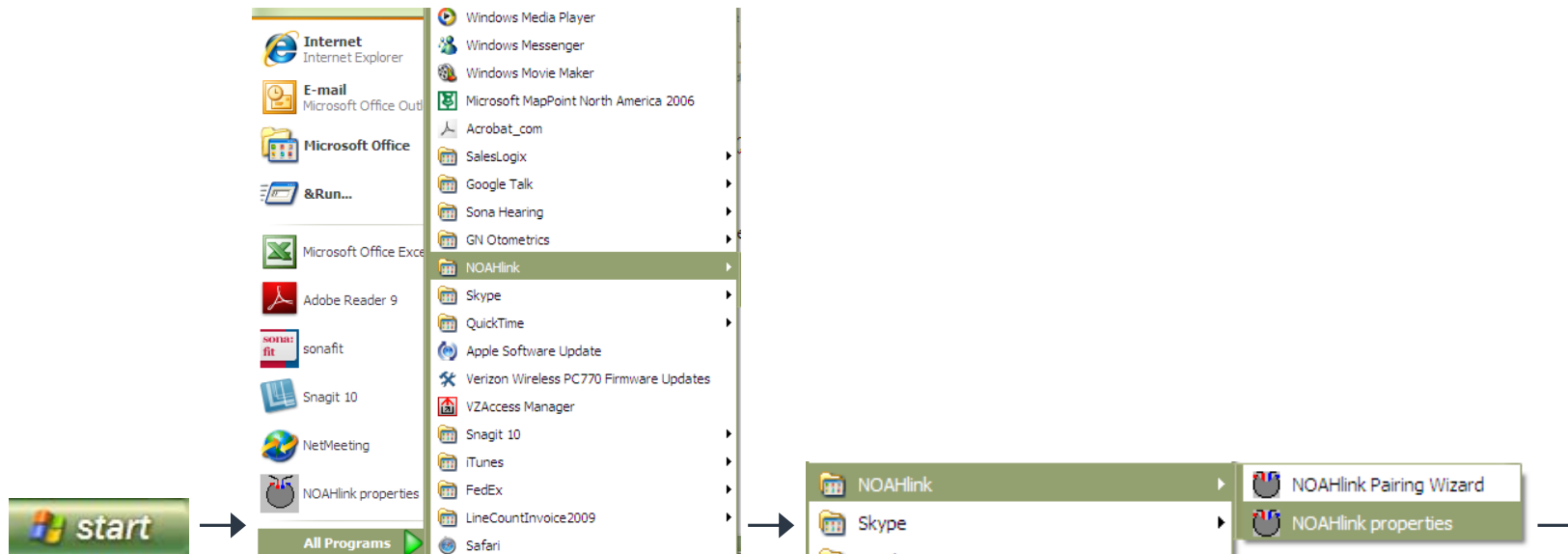


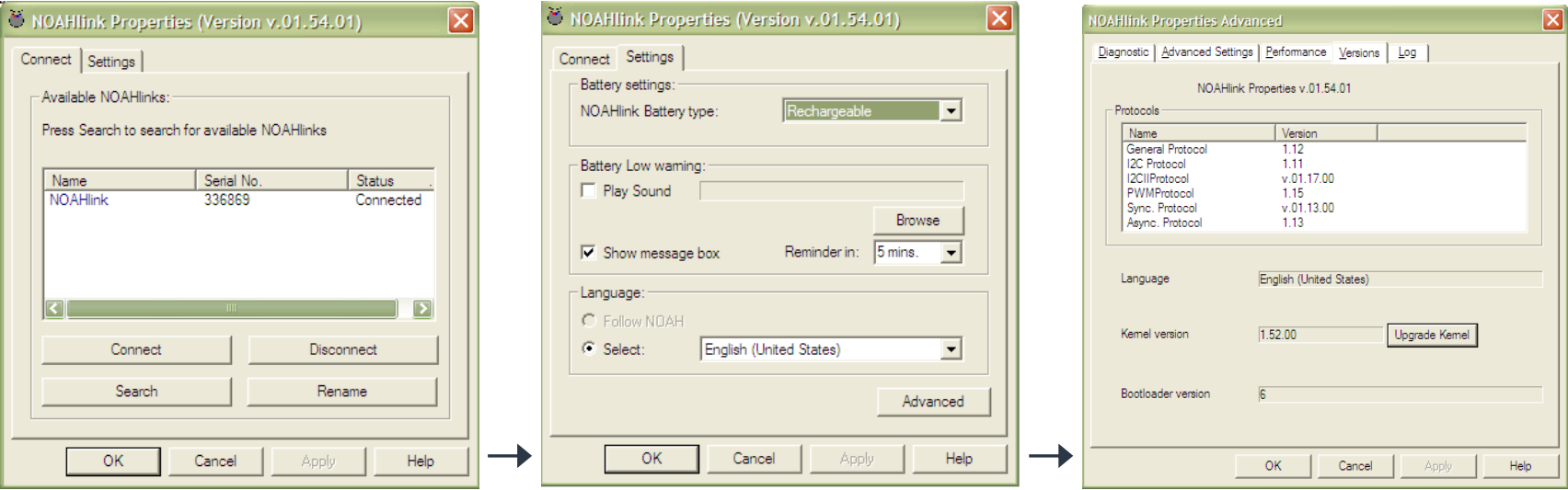

### Quick check for hard disk space:


- : Click Start
- : My Computer
- : Locate Local Disk (C:)
- : Right click on Local Disk (C:)
- : Click Properties
- : General tab will show how much disk space is available

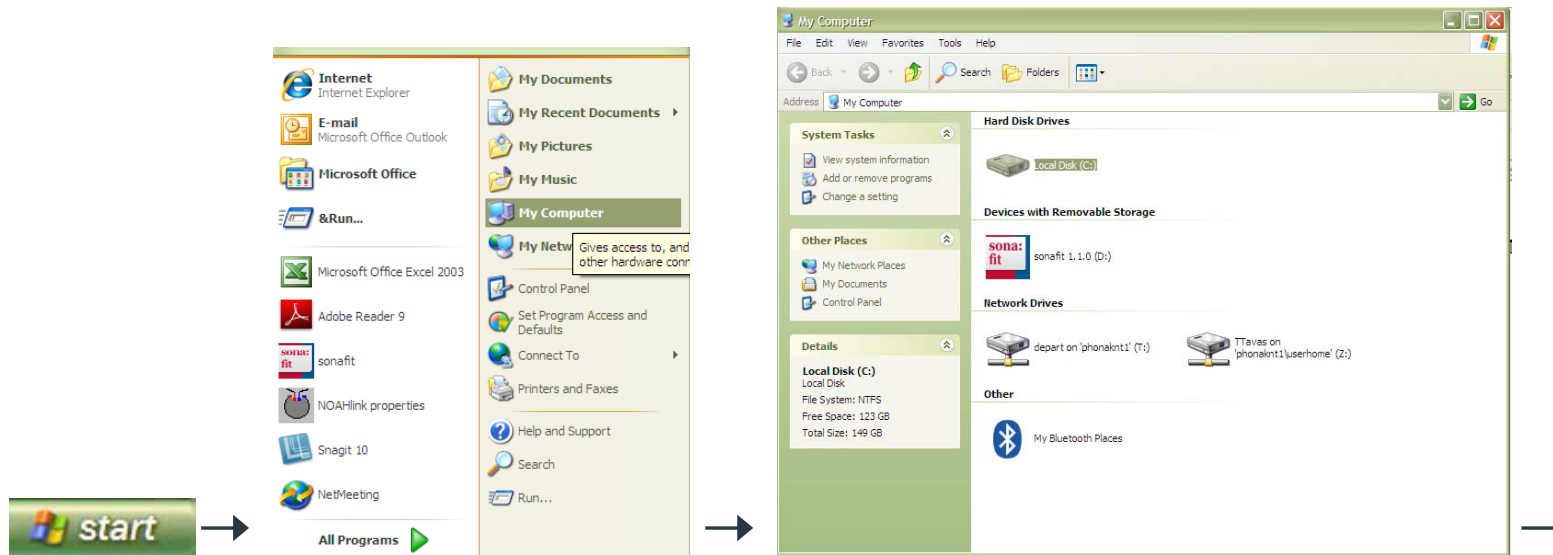

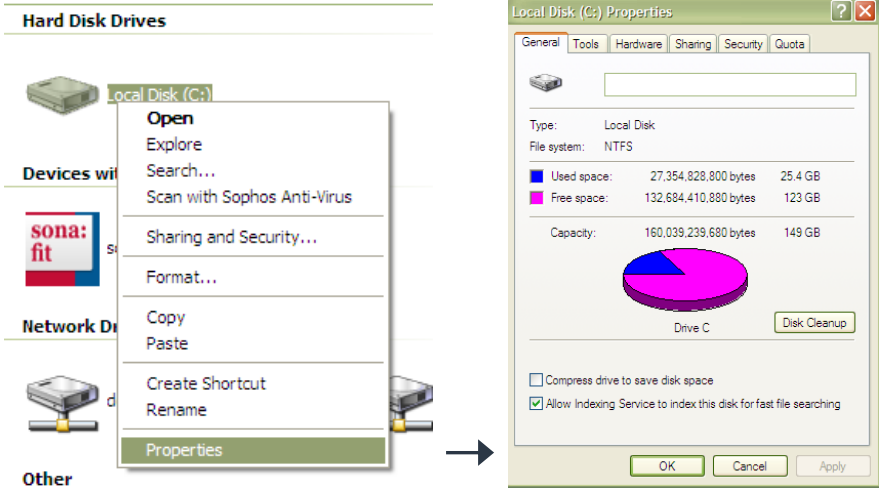

**Quick search for Internet Explorer version:**

- : Open Internet Explorer
- : Click Help
- : Click About Internet Explorer

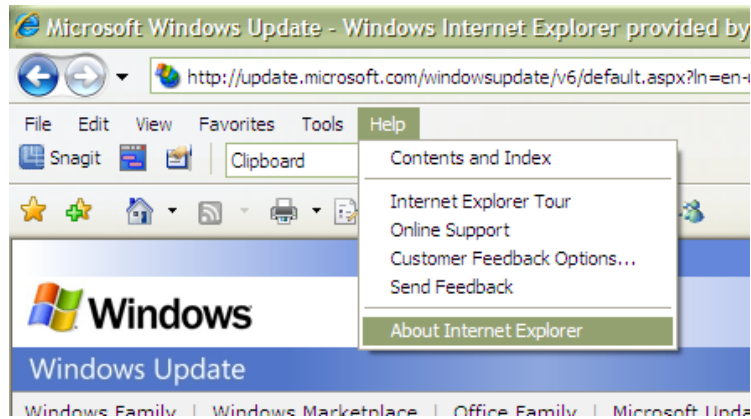

## Quick search for screen resolution

- : Go to Desktop
- : Right Click and go-to Properties
- : Go to Settings tab

